<b>Pulse Oximeter Printer Cable (P/N 036341) Instructions</b>	<b>ENGLISH</b>
<b>Printerkabel pulsoximeter (P/N 036341) Instructies</b>	<b>NEDERLANDS</b>
<b>Câble d'imprimante pour oxymètre de pouls (P/N 036341) Mode d'emploi</b>	<b>FRANÇAIS</b>
<b>Druckerkabel für Pulsoximeter (Teile-Nummer 036341) Gebrauchsanweisung</b>	<b>DEUTSCH</b>
<b>Cavo per stampante di saturimetro (Codice 036341) Foglio di istruzioni</b>	<b>ITALIANO</b>
<b>Cabo da Impressora do Oxímetro de Pulso (N/P 036341) Manual de Instruções</b>	<b>PORTUGUESE</b>
<b>Cable de impresora para pulsioxímetro (P/N 036341) Instrucciones</b>	<b>ESPAÑOL</b>

# Pulse Oximeter Printer Cable (P/N 036341) Instruction Sheet

## Description

The Nellcor<sup>®</sup> pulse oximeter printer cable (part number 036341) is a 1-meter cable that interfaces Nellcor NPB-190, NPB-195, NPB-290, NPB-295, N-395 N-550, N-560, N-595, N-600 and N-600x pulse oximeters with their compatible printers. The printer cable provides a connection between the pulse oximeter RS-232 serial communications port and a serial printer.

## CABLE COMPATIBILITY

The printer cable is compatible with:

- Nellcor NPB-190, NPB-195, NPB-290, NPB-295, N-395 N-550, N-560, N-595, N-600 and N-600x pulse oximeter data communications ports
- RS-232 communications
- 9-pin female d-sub connector


## EXTERNAL CABLE INTERFACE

The printer cable is capable of connecting to:

- Serial printers with 9-pin female d-sub connectors
- Serial printers with 25-pin female d-sub connectors via null 9 to 25 pin serial adapter


## Instructions for Use

- 1) Install two hex standoffs (item 1) on oximeter data port connector (step 2, item 2). Refer to oximeter service manual for oximeter disassembly.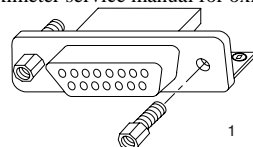

1. Hex Standoff

- 2) Connect printer cable 15-pin connector to pulse oximeter data port connector (item 2).

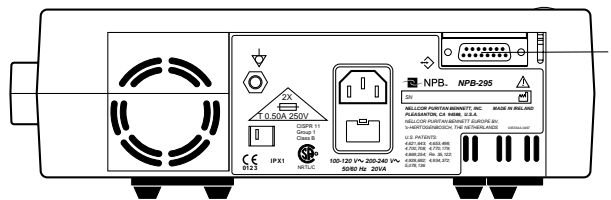2. Data Port Connector

- 3) Connect printer cable 9-pin connector to printer serial port.
- 4) Refer to applicable operator's manual for complete instructions for use, including warnings, cautions, and specifications of the pulse oximeter and printer.

## CLEANING

Clean the printer cable by wiping it with a cleaning solution such as isopropyl alcohol.

**Caution: Do not sterilize by irradiation, steam, or ethylene oxide; do not immerse the cable in cleaning or disinfecting solutions or water.**

---

## WARNINGS

---

- 1) Do not use a damaged cable.
- 2) As with all medical equipment, carefully route patient cabling to reduce the possibility of patient entanglement or strangulation.
- 3) Failure to properly connect the plugs will result in intermittent printing or no printing.
- 4) Cable must only be connected to secondary I/O devices in accordance with IEC 601-1-1 and the resulting system must be verified for proper operation before clinical use.

---

## CAUTION

---

- 1) To avoid damage to the printer cable, grasp the connector rather than the cable when disconnecting from the pulse oximeter or printer.
- 2) Devices connected to cable may emit electromagnetic interference that can interfere with other instruments. Move cable away from the susceptible instruments if interference is suspected.
- 3) Possible interference from sources of electromagnetic interference such as cellular phones, radio transmitters, motors, telephones, high-definition television, lamps, electrosurgical units, defibrillators, and other medical instruments may occur. Keep unit as far away from potential electromagnetic interference sources as possible.
- 4) Cable must not be connected to secondary I/O devices when the cable is not connected to earth reference.
- 5) Cable is not defibrillator proof.

---

## CABLE SPECIFICATIONS

---

### IEC Compatibility

- IEC 601-1: 4.4, IEC 601-1: 4.10, IEC 601-1: 10.2.1, IEC 601-1: 44.5, IEC 601-1: 56.1.b

### Normal Use:

- Ambient temperature: +5° C to +40° C
- Relative humidity: 15% to 95% non-condensing
- Atmospheric pressure: 70 kPa to 106 kPa

### Storage:

- Ambient temperature: -20° C to +60° C
  - Relative humidity: 15% to 95% non-condensing
  - Atmospheric pressure: 50 kPa to 106 kPa
- 

Tyco Healthcare Group LP  
Nellcor Puritan Bennett Division.  
4280 Hacienda Drive  
Pleasanton, CA 94588 USA

Mallinckrodt Inc.  
675 McDonnell Blvd  
P.O. Box 5980  
St. Louis, MO 63134 USA  
Telephone 314.654.2000  
Toll Free 1.800.635.5267

Authorized Representative  
Tyco Healthcare Group UK LTD  
154 Fareham Road  
Gosport PO13 0AS. UK

**Nellcor Puritan Bennett Inc. is a wholly owned subsidiary of Mallinckrodt Inc.**

**To obtain information about warranty, if any, for this product, contact Mallinckrodt's Technical Services Department, or your local Mallinckrodt representative.**

**The following are trademarks of Mallinckrodt Inc.: *Nellcor* and *Nellcor Puritan Bennett*.**

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**Nellcor Puritan Bennett Inc. es una subsidiaria propiedad exclusiva de Mallinckrodt, Inc.**
*Nellcor y Nellcor Puritan Bennett son marcas registradas de Mallinckrodt Inc.*